

DIGITAL inLOOP CONNECTOR - OPTIONAL

OPTICAL



THIS PHOTO IS FOR ILLUSTRATION PURPOSES ONLY. YOUR AUDIO SOURCE WILL PROBABLY BE DIFFERENT.


COAX



If the TV, stereo or cable box has an **OPTICAL** or **COAXIAL (COAX) DIGITAL** output, install the correct cable from the output to the **OPTICAL** or **COAX DIGITAL INPUT**. Do not use more than one input. Cable color may vary.

FINAL STEP

The **VOLUME** control on the **DIGITAL inLOOP CONNECTOR** is only needed for balance if there is an **ANALOG INPUT** in **inLOOP HOME** as well as the **DIGITAL inLOOP CONNECTOR**. Adjust the volume and tone according to the **inLOOP HOME** instructions on the reverse side.

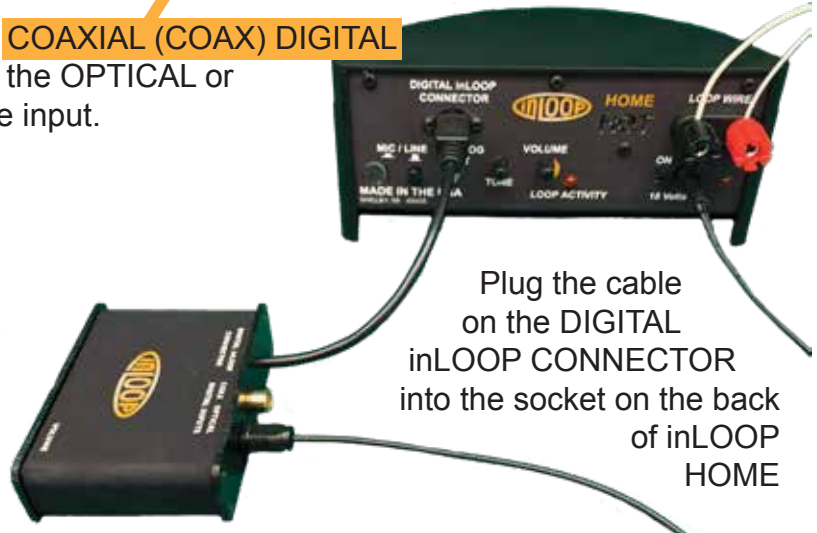

Plug the cable on the **DIGITAL inLOOP CONNECTOR** into the socket on the back of **inLOOP HOME**

PERSONAL inLOOP PAD - OPTIONAL

SECURE THE WIRE SAFELY AROUND THE ROOM. MAKE SURE IT IS NOT A TRIP HAZARD!



PLACE THE PERSONAL inLOOP PAD ON YOUR FAVORITE SEAT OR BEHIND THE HEADREST.

Insert the clips on the end of the wire into the connectors labeled **LOOP WIRE** on the back of **inLOOP HOME**



inLOOP

569 South State Street
Shelby, MI 49455
info@inLOOP.tv

(231) 798- 2399

